



As individual as they are.

GoLife for Men and **GoLife for Women** are as unique in shape and structure as the men and women who wear them.

PHILIPS

RESPIRONICS

sense **and** simplicity

GoLife gives men and women the unique fit each one deserves.

GoLife for Men and GoLife for Women are our first nasal pillows masks to be built around the unique features of men and women. The result is a gender-specific fit that provides tremendous stability.

At first glance it's clear these two masks share a common platform, yet they're quite different. The most obvious distinctions are the attractive colors that differentiate GoLife for Men and GoLife for Women.

But the real beauty is more than skin deep.

A closer look reveals the true design advantages of each. GoLife for Men is designed specifically to fit the larger contours of the male facial structure, while GoLife for Women is developed with a distinctly smaller frame and headgear to more closely fit feminine facial dimensions.

The GoLife platform is built around a simple concept with few parts. The unique soft facial contour arms gently hug the face to maintain stability and a secure seal even as the patient moves. Self-adjusting, optimally angled nasal pillows allow a one-step fit. Comfortable, straightforward headgear comes in two sizes – one for men, the other for women.

Compare GoLife for Men and GoLife for Women and soon you'll discover the unique fit of GoLife masks.

Simple fitting steps

Pull the headgear over your head.
Place the cushion into your nostrils.
Select a cushion that fills your nostrils comfortably without any gaps between the cushion and the walls of your nostrils.



Adjust the side straps.
Do not overtighten.



If needed, adjust the top headgear strap. Allow the headgear to rest just above the ears like a pair of eyeglasses. The back strap will sit at the base of the head.



Connect the tubing to the device.
Turn on the airflow. Make final adjustments while laying down.



To attach a cushion, squeeze the sides so it flexes inward. Place the cushion over the tubing so that it rests on the lower lip of the mask. Push the round nub on one end of the cushion into the circle on the frame and repeat on the other side.




GoLife mask frames and headgear are designed differently to fit men and women.

Soft facial contour arms embrace the face with stability by hugging the cheekbones.



Self-adjusting, optimally angled nasal pillows eliminate the need for manual adjustment and provide a seal that's flexible.

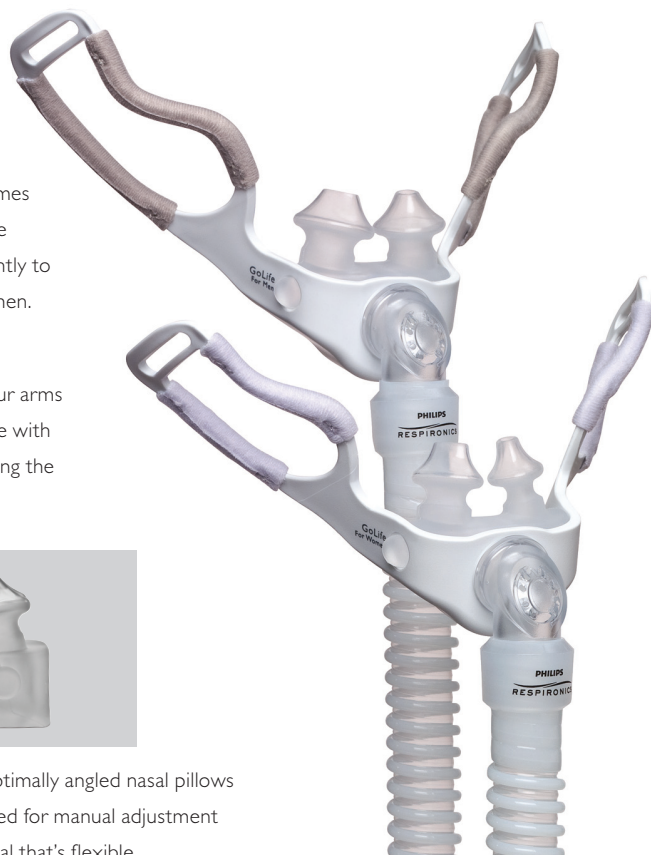

Diffused airflow for quiet exhalation.

Long, multi-directional 18 inch/45.7 cm tube with 360 degree swivel maintains seal during patient movement.



GoLife for Women has smaller headgear that sits lower on the back of the head to prevent slippage and to help provide a better fit.

Ordering information		
GoLife for Women mask	Part number	U.S. HCPCS code
FitPack: mask with headgear plus petite, small, and medium cushions	1082577	A7034, A7035
FitPack: mask only plus petite, small, and medium cushions	1082579	A7034
Mask subassembly without cushions or headgear	1082578	*
Mask subassembly (frame only, no swivel tubing)	1082583	N/A
GoLife for Women headgear	1082581	A7035
GoLife for Men mask	Part number	U.S. HCPCS code
FitPack: mask with headgear plus small, medium, and large cushions	1073114	A7034, A7035
FitPack: mask only plus small, medium, and large cushions	1073329	A7034
Mask subassembly without cushions or headgear	1073328	*
Mask subassembly (frame only, no swivel tubing)	1078408	N/A
GoLife for Men headgear	1073336	A7035
Replacement parts and accessories	Part number	U.S. HCPCS code
Replacement cushion, petite	1073332	A7033
Replacement cushion, small	1073333	A7033
Replacement cushion, medium	1073334	A7033
Replacement cushion, large	1073335	A7033
Tube management clip, 2 per pack	1073337	N/A
Swivel tubing (18 inch/45.7 cm) with exhalation port	1078407	A7045

*Requires addition of mask cushion for HCPCS A7034

**Philips Healthcare is part of
Royal Philips Electronics**

How to reach us

www.philips.com/healthcare
healthcare@philips.com

Asia
+49 7031 463 2254

Europe, Middle East, Africa
+49 7031 463 2254

Latin America
+55 11 2125 0744

North America
+1 425 487 7000
800 285 5585 (toll free, US only)

Philips Respironics
1010 Murry Ridge Lane
Murrysville, PA 15668

Customer Service
+1 724 387 4000
800 345 6443 (toll free, US only)

Philips Respironics International
Headquarters
+33 1 47 28 30 82

Philips Respironics Asia Pacific
+65 6882 5282

Philips Respironics Australia
+61 (2) 9666 4444

Philips Respironics China
+86 021 24127311

Philips Respironics Deutschland
+49 8152 93 06 0

Philips Respironics France
+33 2 51 89 36 00

Philips Respironics Italy
+39 039 203 1

Philips Respironics Sweden
+46 8 120 45 900

Philips Respironics Switzerland
+41 6 27 45 17 50

Philips Respironics United Kingdom
+44 800 1300 845

www.philips.com/respironics

Respironics, FitPack, and GoLife are trademarks of Respironics, Inc. and its affiliates. All rights reserved.

Please visit www.philips.com/golife



© 2011 Koninklijke Philips Electronics N.V.
All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.
Geyer JJ 05/11/11 MCI 4104142 PN 1082047