


# Instructions

## AG INDUSTRIES

### CPAP Tube Cleaning Wand

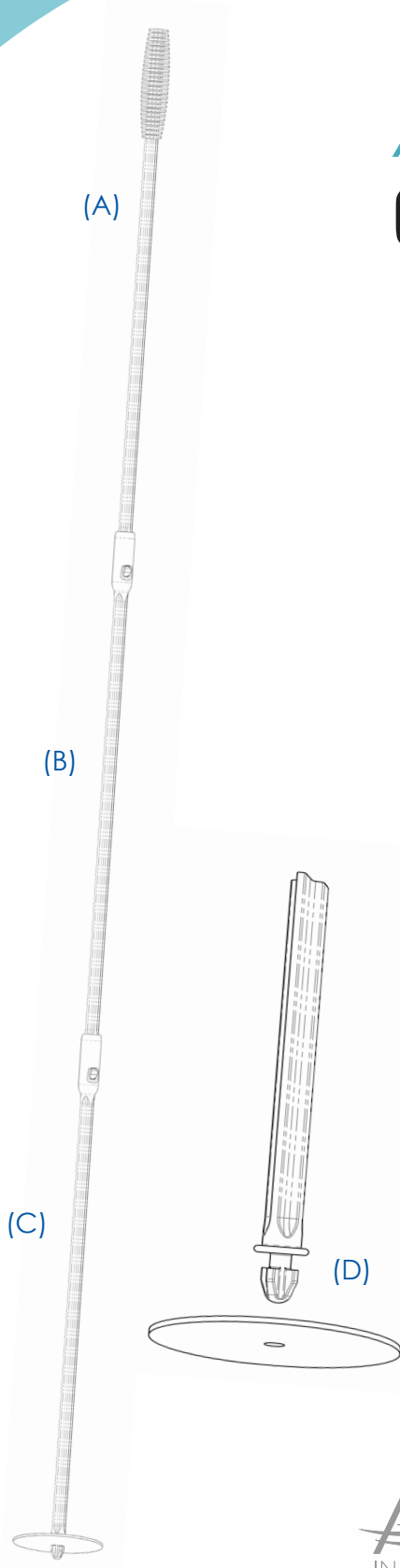

- 1) Holding the handled wand section (A) by the ribbed handle, slide the small button on opposite end into the receiving hole of the center wand section (B) until they snap in place. Next, slide the small button on the end of the center section into the receiving hole of the third wand section (C) until they snap in place.
- 2) Place a single cleaning pad over the tapered end of the wand (D).
- 3) Disconnect CPAP tube from machine and mask.
- 4) Insert wand into CPAP tube and slide back and forth in short motions down the length of the tube until the wand is fully inserted.
- 5) Remove wand and repeat from the other end of the tube.
- 6) When cleaning is complete, remove and discard cleaning pad. Then disconnect wand sections by depressing the small button at each joint and slide apart. Store in provided protective bag.

AG Industries recommends cleaning your CPAP tube daily after each use of your CPAP machine.

Please contact your home medical equipment supplier for replacement cleaning pads (AG-CLN-WAREP) and be sure to look for additional AG Industries CPAP Supplies!



Please contact customer service or your sales representative for more information on this product.